

Reprocessed for



Sustainability Solutions

Instructions for Use Reprocessed Freedom Splint

Reprocessed Device for Single Use. Reprocessed for Stryker Sustainability Solutions.

Caution: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

NON-STERILE NOT MADE WITH NATURAL RUBBER LATEX

Explanation of Icons



Date of Reprocessing



Use by Date



Product Code



Do Not Reuse



See Instructions For Use

Indications for Use:

- The Freedom splint is an alternative to limb holders that may be used as a hand, wrist, elbow, and knee splint or an I.V. arm board.
- The Freedom splint helps prevent tube pulling and aids I.V. therapy to promote healing.
- The Freedom splint is ideal for prevention and release of contractures.

Instructions for Use:

1. Loosen the adjustment straps and open the Splint.
2. Make sure that the plastic buckles are positioned towards the patient.
3. Fasten the splint on the desired position of the limb. It should be centered over the elbow so that the opening is towards the inside of the arm. For leg use, the splint should be centered over the knee with opening towards the front of the leg.
4. Fasten the splint with hook – and-loop strap through the plastic buckle securing back onto itself. To secure an infant sized splint, wrap the hook-and-loop strap around the limb and attach to the corresponding hook-and-loop strap. Leave enough room so that two fingers can be inserted between the device and the patient's limb to ensure adequate circulation.
5. If the splint size needs to be reduced, fold the outside panel with hook-and-loop strap towards the inner lining, minimizing the splint width.

Reprocessed Freedom Splint

Warnings:

1. Skin Conditions should be monitored frequently.
2. Do not over tighten or impair circulation.

Adult Freedom Splint

OEM Catalog #	Length	Limb Circumference
8168XS	10"	3-12"
8168S	11"	3-14"
8168M	13"	6-17"
8168L	14"	8-21"
8169	18"	14-27"

Infant and Pediatric Freedom Splints

OEM Catalog #	Length	Limb Circumference
8168I	3"	1 ½-4"
8168IL	4 ½"	1 ½-4"
8168P	7"	3-10"

The OEM information listed on the label is provided as device ID prior to reprocessing and may contain the trademarks of unrelated third parties that do not sponsor this device.