



Sustainability Solutions





Instructions for Use Reprocessed Blood Pressure Cuff

Reprocessed Device for Single Use. Reprocessed for Stryker Sustainability Solutions.

Caution: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

NON-STERILE NOT MADE WITH NATURAL RUBBER LATEX

Explanation of Icons

	Date of Reprocessing
	Use by Date
REF	Product Code
	Do Not Reuse
	See Instructions For Use

Indications for Use

Clinical applications for the Blood Pressure Cuffs are as follows:

- Used to monitor the vital signs of the patient

Cautions

- Proper cuff application is essential.
- Cuff should be removed immediately if the patient experiences tingling, numbness, or pain. The physician should be notified.
- Cuff should not be used over an open wound, gangrene, untreated infected wound, recent skin grafts, or dermatitis.
- Cuff should not be inflated too tightly so as not to occlude circulation.
- If inflated too loosely, the cuff will not deliver maximum benefits.

Contraindications

The Cuff should NOT be used in the following conditions:

1. Any local condition in which the garment would interfere:

- open wounds
- untreated infected wounds
- gangrene
- recent skin graft
- dermatitis

Reprocessed Disposable Infusor Bag

Directions for Use

1. Place cuff under upper arm of patient.
2. Fold Velcro side of cuff over the first side. Secure Velcro® attachment.
3. Ensure that the tubes are directed down away from the cuff.
4. Ensure that the cuff is securely attached to the patient.
5. Inflate cuff.

The OEM information listed on the label is provided as device ID prior to reprocessing and may contain the trademarks of unrelated third parties that do not sponsor this device.

This product and its packaging have been exposed to ethylene oxide gas (EtO). Even though the product is processed in compliance with all applicable laws and regulations relating to EtO exposure, proposition 65, a State of California voter initiative, requires the following notice:

Warning: this product and its packaging have been exposed to ethylene oxide. The packaging may expose you to ethylene oxide, a chemical known to the State of California to cause cancer or birth defects or other reproductive harm.