



Sustainability Solutions

Instructions for Use Reprocessed ECG Leads

Reprocessed Device for Single Use. Reprocessed for Stryker Sustainability Solutions.

Caution: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

NON-STERILE NOT MADE WITH NATURAL RUBBER LATEX

Explanation of Icons



Date of Reprocessing



Use by Date

REF

Product Code



Do Not Reuse



See Instructions For Use

Indications for Use

The Stryker Sustainability Solutions Reprocessed ECG Leads is a non-invasive device to provide continuous electrocardiograph (ECG) monitoring by transmitting signals from patient electrode pads to the ECG monitor. The Reprocessed ECG Leads meets the requirements of CFR Title 21 Part 898 and the ANSI/AAMI Standard, ECG Cables and Leads (ANSI/AAMI EC53-1995).

Warnings

- Do not immerse in water or cleaning solutions.
- Do not use the leads during MRI scanning.

Directions for Use

1. Ensure that the proper reusable adapter is plugged into the monitor.
2. Remove leads set from bag and plug the connector into the adapter.
3. Peel the leads as needed and connect the leads connectors to the appropriate electrodes by pushing the pink button and securing the connection.
4. If the product becomes soiled, clean with mild soap and water.

Reprocessed ECG Leads Products Offered

ECG Leads & Cables	3M Healthcare	YMDLW3P	3 Lead Shielded Ribbon Cable, AMC&E DIN to Pinch
ECG Leads & Cables	3M Healthcare	YMDLW3S	3 Lead Shielded Ribbon Cable, AMC&E DIN to Snap
ECG Leads & Cables	3M Healthcare	YMDLW5P	5 Lead Shielded Ribbon Cable, AMC&E DIN to Pinch
ECG Leads & Cables	3M Healthcare	YMDLW5S	5 Lead Shielded Ribbon Cable, AMC&E DIN to Snap
ECG Leads & Cables	3M Healthcare	YMDLW5SDSP	5-Lead Shielded Ribbon Cable, AMC&E DataScope TelePak-Pinch
ECG Leads & Cables	3M Healthcare	YMDLW5SGEP	5 Lead Shielded Ribbon Cable, AMC&E GE Grouped o Pinch
ECG Leads & Cables	3M Healthcare	YMDLW5SP	5 Lead Shielded Ribbon Cable, AMC&E DIN to Pinch w/ Shower Shield
ECG Leads & Cables	3M Healthcare	YMDLW6SNKP	6-Lead Shielded Ribbon Cable, AMC&E Nihon Kohden to Pinch
ECG Leads & Cables	AMC&E	LW-209DS50/3A	3-lead Leadset, Shielded, DIN to Snap
ECG Leads & Cables	AMC&E	LW-209DS50/5A	5-lead Leadset, Shielded, DIN to Snap
ECG Leads & Cables	AMC&E	LW-309DS50/3A	3-lead Leadset, Shielded, DIN to Pinch
ECG Leads & Cables	AMC&E	LW-309DS50/5A	5-lead Leadset, Shielded, DIN to Pinch
ECG Leads & Cables	AMC&E	LW-309DS50/5AT	5-lead Telemetry Set, Shielded, DIN to Pinch w/ Shower Shield
ECG Leads & Cables	AMC&E	LW-341DS50/5A	5-Lead Grouped Set, Shielded, Pinch (Grabber) to Multi-Link
ECG Leads & Cables	AMC&E	LW-341DS50/5AT	5-Lead Telemetry Leadset, Pinch (Grabber) to Multi-Link
ECG Leads & Cables	AMC&E	LW-381DS50/6A	6-Lead Grouped Set, Shielded, Pinch to Molded Nihon Kohden
ECG Leads & Cables	AMC&E	LW-391DS50/5A	5-Lead Grouped Set, Shielded, Pinch (Grabber) to Telepack
ECG Leads & Cables	Covidien	33103	Kendall 3 Lead Cable & Wire,
ECG Leads & Cables	Covidien	33112	Kendall 5 Lead, GE Telemetry,
ECG Leads & Cables	Covidien	33113	Kendall 5 Lead V Philips Telemetry,
ECG Leads & Cables	Covidien	33114	Kendall Philips Telemetry Lead,
ECG Leads & Cables	Covidien	33125	Kendall 6 Lead Telemetry, GE Apex Pro CH
ECG Leads & Cables	Covidien	33126	Kendall 6 Lead Telemetry, GE Apex Pro FH
ECG Leads & Cables	Covidien	33132	Kendall 5 Lead Telemetry, Draeger
ECG Leads & Cables	Covidien	33133	Kendall 5 Lead Telemetry, Spacelabs
ECG Leads & Cables	Covidien	33134	Kendall 6 Lead Telemetry, Nihon Kohden
ECG Leads & Cables	Covidien	33136R36	Kendall 5 Lead Radiolucent 3',
ECG Leads & Cables	Covidien	33135T	Kendall DL Disposable Dual Connect 5 Lead Cable and Lead
ECG Leads & Cables	Covidien	33136T	Kendall DL Disposable Dual Connect Lead Set
ECG Leads & Cables	Covidien	33145T	Kendall DL Disposable Dual Connect Cable
ECG Leads & Cables	Curbell Medical Products Inc	DLP-05AB-40AS-0100	5 Lead Set (Twin Pin), Snap
ECG Leads & Cables	Curbell Medical Products Inc	DLP-05AD-40AM-0100	5 Lead Set (Multi Link), Snap
ECG Leads & Cables	Curbell Medical Products Inc	DLP-05BB-40AS-0100	5 Lead Set (Twin Pin), Pinch
ECG Leads & Cables	Curbell Medical Products Inc	DLP-05BD-40AM-0100	5 Lead Set (Multi Link), Pinch
ECG Leads & Cables	GE Healthcare	394111-002	3-Lead Leadwire Set, Telemetry, Snap
ECG Leads & Cables	GE Healthcare	394111-006	5-Lead Leadwire Set, Telemetry, Snap with 3/2 Diversity
ECG Leads & Cables	GE Healthcare	394111-008	6-Lead Leadwire Set, Telemetry, Snap 3/3 Diversity
ECG Leads & Cables	Philips Healthcare	M1622A	3 Lead Set Philips IntelliVue, ICU Clip, 0.45m
ECG Leads & Cables	Philips Healthcare	M1624A	3 Lead Set Philips IntelliVue, ICU Clip, 0.7m
ECG Leads & Cables	Philips Healthcare	M1644A	5 Lead Set Philips IntelliVue, ICU Snap, Limb, 1.0m/1.6m
ECG Leads & Cables	Philips Healthcare	M1647A	5 Lead Set Philips IntelliVue, ICU Miniclip, Limb, 0.7m/1.3m
ECG Leads & Cables	Philips Healthcare	M1671A	3 Lead Set Philips IntelliVue, ICU Grabbers, 1m

Reprocessed ECG Leads Products Offered

ECG Leads & Cables	Philips Healthcare	M1673A	3 Lead Set Philips IntelliVue, ICU Snaps, 1m
ECG Leads & Cables	Philips Healthcare	M1675A	3 Lead Set Philips IntelliVue, OR Grabbers, 1m
ECG Leads & Cables	Philips Healthcare	M1680A	6 Lead Set Philips IntelliVue, ICU Grabber, 1.0m/1.6m
ECG Leads & Cables	Philips Healthcare	M1682A	6 Lead Set Philips IntelliVue, ICU Snap, 1.0m/1.6m
ECG Leads & Cables	Philips Healthcare	M1684A	6 Lead Set Philips IntelliVue, OR Grabber, 1.0m/1.6m
ECG Leads & Cables	Philips Healthcare	M1968A	5 Lead Set Philips IntelliVue, ICU Grabber Limb, 1.0m/1.6m
ECG Leads & Cables	Philips Healthcare	M1973A	5 Lead Set Philips IntelliVue, OR Grabber, Limb, 1.0m/1.6m

The OEM information listed on the label is provided as device ID prior to reprocessing and may contain the trademarks of unrelated third parties that do not sponsor this device.

This product and its packaging have been exposed to ethylene oxide gas (EtO). Even though the product is processed in compliance with all applicable laws and regulations relating to EtO exposure, proposition 65, a State of California voter initiative, requires the following notice:

Warning: this product and its packaging have been exposed to ethylene oxide. The packaging may expose you to ethylene oxide, a chemical known to the State of California to cause cancer or birth defects or other reproductive harm.